## One Piece Clinical Grade Blood Pressure Cuffs

Index Line.

Cuff index line must fall within range markings.

Artery symbol and arrow should be placed over brachial or femoral artery.

Not made with PVC.

Not made with natural rubber latex.

Product in compliance with Council Directive 93/42/EEC Medical Devices Directive

Symbol indicating arm circumference

Symbol indicating lot code of cuff

Symbol indicating Caution

Symbol indicating manufacturer

Symbol indicating to refer to "Instructions for Use"

Indicates device safe only by or on the order of a licensed practitioner

Indicates keep dry

## RANGES / COLORES

Size:	Range:
Infant	8-13 cm
Child	12-19 cm
Sm Adult	17-25 cm
prmeCUFF S	Royal Blue 18-25 cm
prmeCUFF S	Green 17-25 cm
Adult	Navy Blue 23-33 cm
prmeCUFF	Navy Blue 23-33 cm
prmeCUFF	Blue 28-40 cm
Lg Adult	Burgundy 31-40 cm
prmeCUFF XL	Burgundy 40-55 cm
Thigh	Brown 38-50 cm

## WARNINGS & CONTRAINDICATIONS

Use of the cuff can temporarily cause loss of function if simultaneously used with monitoring equipment on the same limb.  
Replace cuff when worn or damaged.  
Federal (US) law restricts this device to sale by or on the order of a physician.  
Cuffs are intended for use with non-invasive blood pressure monitors only.

Follow the application instructions for use to ensure the correct size cuff for the patient. Failure to do so will adversely affect the accuracy of the reading.  
If user lock connectors are used in the construction of tubing, there is a possibility to connect the cuff to intravenous fluids, allowing air to be pumped into a blood vessel, potentially causing serious injury.  
Minimize limb movement during the measurement.

Apply the cuff, other than that of the patient's limb, while measurement is in progress. Do not compress or pinch cuff tubing.  
Ensure patient's skin is intact before applying the cuff.  
If patient has a mastectomy, place the cuff on the arm opposite the site of the mastectomy. If a patient has a double mastectomy, the cuff must be placed on the least dominant arm.

therapy, or an arterio-venous (A-V) shunt, due to the presence of temporary interference of blood flow, do not apply the cuff on that arm. Potential injury to patient may occur.

The use of cuff while using other monitoring medical equipment could affect the accuracy or operation of said equipment.  
Promptly remove cuff from patient when monitoring is not in progress.

During frequent or extended measurement, check cuff site and limb to ensure proper blood flow. Do not apply cuff where circulation may be compromised.

## APPLICATION

1. Wrap the cuff around the patient's upper arm midway between the elbow and the shoulder.

2. Ensure the **ARTERY** arrow is over the brachial artery, between the bicep and tricep muscles on the inside of the arm.

3. Use the **RANGE** indicator with the **INDEX** line to check that the arm falls within the specified range of that cuff. Use appropriate larger or smaller cuff if necessary.

4. Wrap the cuff snugly around the arm.

## CLEANING INSTRUCTIONS

The following cleaning methods have been applied 20 times to the cuff without any apparent negative effects.

The cuff may be sprayed with a mild disinfectant solution (e.g. Cidezyme<sup>®</sup>, ENZO<sup>®</sup>, or 10% bleach solution), rinsed with distilled water and line dry. Ensure that no liquid enters tubing.

OR

The cuff may be wiped down with a mild disinfectant wipe (e.g. Sanit-Cloth<sup>®</sup>) and line dry.

## ENVIRONMENTAL CONDITIONS

Operating Ranges
Temp: 0 - 50°C
RH: 15 - 95% non-condensing
Storage Ranges
Temp: -20 - 65°C
RH: 15 - 95% non-condensing

## WARRANTY

SunTech Medical warrants our blood pressure cuff products to be free from defects in material and workmanship 24 months from the original date of purchase. This limited warranty covers the no charge replacement of the cuff under normal wear and tear conditions when returned to the attention of Service Department at the address below depending on location.

Contact the Service Department at either of these locations to receive a Return Material Authorization number before sending any product. Cuffs should be returned to the attention of the appropriate Service Department at the addresses below.

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SunTech Medical, Inc.
SunTech Medical, Ltd. Europe