Vital Signs™ English

ECG Leadwires for KISS™ System

Instructions for Use

| | |
|------------|---|
| 30401590 | Vital Signs™ Set of 10 Leadwires IEC for KISS™ System |
| 30343374 | Vital Signs™ Leadwire L for KISS™ System |
| 30343371 | Vital Signs™ Leadwire R for KISS™ System |
| 30343366 | Vital Signs™ Leadwire Cz for KISS™ System |
| 30343367 | Vital Signs™ Leadwire C3 for KISS™ System |
| 30343368 | Vital Signs™ Leadwire C4 for KISS™ System |
| 30343373 | Vital Signs™ Leadwire A for KISS™ System |
| 30343375 | Vital Signs™ Leadwire V for KISS™ System |
| 202262-006 | Vital Signs™ Leadwire A1 for KISS™ System |
| 202262-007 | Vital Signs™ Leadwire A2 for KISS™ System |
| 202262-009 | Vital Signs™ Leadwire A3 for KISS™ System |


Customer Support: customersupport@vyaire.com

1000043089A / 2019-11

| | |
|---|--|
| REF | Caution Non-Sterile |
| LOT | Expiration Date |
| X | Quantity |
| M | Date of Manufacture |
| Manufacturer | Not made with natural rubber latex |
| Temperature Limit | Follow Instructions for Use |
| MD | Medical Device |
| W | Water/Electric/Electrostatic Equipment |
| UDI | Unique Device Identifier |
| CE | MR Unsafe |
| Current Limiting Devices Present | |

Intended Use

The leadwires are reusable and are used to deliver the ECG signals from the suction electrodes to the KISS and KISS Multilead System. They are also used to provide air-pressure to the suction electrodes.

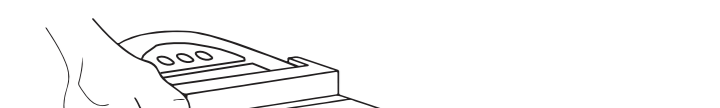Instructions for use

This document contains the instructions necessary to assemble, disassemble and clean the leadwires.

The leadwires are attached to the ECG electrodes and are not replaced. They are supplied including the air and tear and, therefore can be replaced. The air valve is integrated in the connector that is plugged into the distributor. The connector of the damaged leadwire has a slight tear.

- Remove the damaged leadwire from the electrode distributor, grasping the connector directly at the air valve.
- Remove the connector of the new leadwire into the electrode distributor. Always hold connector in the correct. Make sure that it is properly inserted, otherwise the connector will not be recognized by the system.
- Connect the electrode to the new leadwires.

For further instructions on Suction Electrodes and compatibility information, see the device operator’s manual.

Any safety incident that has occurred in relation to the device should be reported to the manufacturer at customersupport@vyaire.com and the competent authority of the Member State in which the patient is established.

Maintenance

Before use, check that the product is intact and clean. Maintenance has successfully tested the enclosed processing instructions for 80 cycles.

Warnings

MR unsafe. Do not use this product in MRI environment.

Ensure proper contact of the electrolyser unit neutral electrode to the patient to avoid burn risk. Do not use the device if you do not comply with the validated parameters during every processing.

Use appropriate placement and safe functioning of the product, check stability of connection with monitor and/or accessories initially and occasionally thereafter.

Check appropriate placement or position of the product initially and occasionally or time to prevent entanglement, pressure, tripping, etc. or when you might cause any obstruction, irritation, decubiti, falling or other items or people, etc.

Disconnect the leadwire from system components before cleaning and disinfecting. Do not clean and disinfect the connector of leadwire that is plugged into the electrode distributor.

Pre-Processing Instructions

Instructions for use are followed prior to cleaning and disinfecting.

1. Initial cleaning and disinfection of device as soon as possible after use (recommended within 1 hour).

2. Transport devices using the institution’s established transport procedure. Use of a pressure bag is as possible after use by wiping the device.

3. Device shall be cleaned and disinfected utilizing a minimum of one (1) cycle of the disinfection method described in section 2.

Cleaning Instructions

1. Ensure all pre-processing instructions are followed prior to cleaning.

2. Prepare the enzymatic / neutral pH detergent, utilizing tap water containing a temperature range of 27°C to 44°C (81°F to 111°F), per manufacturer’s instructions.

3. Use a soft, clean cloth saturated with the cleaning solution to manually or lightly scrub the device for a minimum of 5 minutes. In conjunction with this step, use an appropriately sized brush (see manual) to scrub the device for a minimum of 5 minutes.

4. Using a sterile, soft cloth saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat two (2) to three (3) times for a total of three (3) rinse cycles, utilizing fresh purified water each time.

5. Dry the device using a sterile, lint-free cloth.

Intermediate level disinfection instructions (option 2)

1. Prepare a 2% Glutaraldehyde disinfectant solution (CIDEX® or equivalent) according to the manufacturer’s directions.

2. Using a sterile, soft cloth saturated with the disinfection solution, manually disinfed all contaminated surfaces of the device for a minimum of 15 minutes.

3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lamun brush) to disinfed hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of the disinfection step.

4. Using a sterile, soft cloth saturated with purified water (PURW) at 27°C – 44°C, manually rinse by wiping the device for 30 seconds. Repeat two (2) to three (3) times for a total of three (3) rinse cycles, utilizing fresh purified water each time.

5. Dry the device using a sterile, lint-free cloth.

Storage

Storage Temperature: -30°C/22°F to +60°C/140°F

Disposal

Reprocess products prior to disposal. Dispose electrical and electronic devices in compliance with WEEE or disposal local and state environmental regulations. This product does not contain PVC.

Technical description

This product is classified as a defibrillation-proof type CF applied part.

For technical description see the host device user guide or manual, or contact your local sales representative.

Not made with natural rubber latex.

Ordering Information

For further information, please contact your local sales representative or visit vyaire.com.

Trademarks are the property of their respective owners.

©2019 Vyaire, Vyaire, the Vyaire logo and Vital Signs are trademarks or registered trademarks of Vyaire Medical, Inc. or one of its affiliates.

Vyaire Medical Oy
Pöytäkatu 1
FI-00510 Helsinki, Finland
+358-9-3231-3264

Customer Support: customersupport@vyaire.com

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