## Leadwire Clip Adapter for Adhesive Electrodes for use in KISS™ System

<span></span>			
3034421	Vital Signs™ Diagnostic Cardiology Adapter, KISS™ system to disposable electrodes (leadwire)	<span></span>	
3840184	Vital Signs™ Diagnostic Cardiology Adapter Set, 4mm Banana Socket to Grabber, 10mm	<span></span>	
<b>REF</b>	Catalogue Number	<span></span>	Caution Non-sterile
<b>LOT</b>	Lot Number	<span></span>	Date of Manufacture
<b>X</b>	Quantity	<span></span>	Not made with natural rubber latex.
<span></span>	Manufacturer	<span></span>	Follow Instructions for Use
<span></span>	Temperature Limit	<span></span>	MRI Unsafe
<b>MD</b>	Medical Device	<span></span>	Unique Device Identifier
<span></span>	Waste Electrical & Electronic Equipment	<span></span>	

**Intended use**
The reusable Clip is used to connect an adhesive electrode to a leadwire for recording the ECG signal to the KISS™ or KISS Multilead System.

ECG adapters provide connection between ECG surface electrodes and ECG leadwires to transmit electrical signal for diagnostic and monitoring purposes.

Use is limited by the indicators for use of the connected monitor and diagnostic equipment.

### Instructions for use

Always contain the instructions necessary to assemble and disassemble the KISS Clip before the KISS Leadwire and adhesive electrode. For further instructions on KISS leadwires, KISS Clip and adhesive electrodes, as well as compatibility information, see the device operator's manual.

Connect the leadwire to the metallic conus contact pin of the clip. Press clip firmly into the leadwire.
1. Prepare the skin. Shave any hair from the selected electrode sites, and clean and dry the skin.
2. Position and apply disposable electrode on patient skin site.
3. Connect the white clip to the adhesive electrode.
4. Remove the disposable electrode.

- These products are indicated for use by qualified medical personnel only.
- These products are not intended for use for diagnostic or detailed information.
- For use with GE Healthcare equipment.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer at GMR-Res-Complaint-Intake@vyaire.com and to the regulatory authority of the Member State in which the user and/or patient is established.

**Maintenance**
Before use, check that the product is intact and clean. Manufacturer has successfully tested the enclosed reprocessing instructions for use with the following reprocessing methods:

**Warnings**
-MRI Unsafe. Do not use this product in MRI environment.
- Do not let the conductive product parts contact any other conductive parts or the patient.
- Ensure proper contact of the electrocary unit neutral electrode to the patient to avoid burns at monitor measurement sites. Do not place product over the electrocary unit neutral electrode or electrocary cables.
- Do not allow humidity or liquid to enter the product.

- Do not use the product if you observe any signs of damage. Do not use product if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Do not use product if you see any signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Products are supplied non-sterile and must be unpacked, cleaned and disinfected prior to first use.
- Adequate cleaning is a necessary requirement for subsequent effective disinfection.

- Follow the equipment and product-specific validated procedures for cleaning and disinfection.
- Use the equipment and accessories must be maintained and checked regularly according to the manufacturer's instructions.
- Additionally all applicable laws, standards and guidelines for reprocessing equipment should be followed.
- Use of personal protective equipment is the responsibility of the user.
- Deviation from the defined procedures within this instruction is the responsibility of the user.
- Manufacturer is not responsible for damages resulting from improper reprocessing. It rests with the user to validate their procedures and to comply with all applicable regulatory requirements.
- Use of accessories with the product is not recommended. Using accessories with the product other than the ones referenced might result in a measurement error or hazardous condition. Observe connector part color coding and/or markings on products to ensure adequate compatibility with monitor and other accessories.

- Do not use inappropriate and safe functioning of the product, check stability of connection with monitor and/or accessories initially and occasionally over time.
- Do not use inappropriate position or position of the product initially and occasionally over time to prevent entanglement, pressure, tripping, etc., to occur as they might cause injury, obstruction, irritation, decubitus, falling or other safety hazards.
- Do not use the product if you observe any signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Do not use product if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Products are supplied non-sterile and must be unpacked, cleaned and disinfected prior to first use.
- Adequate cleaning is a necessary requirement for subsequent effective disinfection.

- Follow the equipment and product-specific validated procedures for cleaning and disinfection.
- Use the equipment and accessories must be maintained and checked regularly according to the manufacturer's instructions.
- Additionally all applicable laws, standards and guidelines for reprocessing equipment should be followed.
- Use of personal protective equipment is the responsibility of the user.
- Deviation from the defined procedures within this instruction is the responsibility of the user.
- Manufacturer is not responsible for damages resulting from improper reprocessing. It rests with the user to validate their procedures and to comply with all applicable regulatory requirements.
- Use of accessories with the product is not recommended. Using accessories with the product other than the ones referenced might result in a measurement error or hazardous condition. Observe connector part color coding and/or markings on products to ensure adequate compatibility with monitor and other accessories.

- Do not use inappropriate and safe functioning of the product, check stability of connection with monitor and/or accessories initially and occasionally over time.
- Do not use inappropriate position or position of the product initially and occasionally over time to prevent entanglement, pressure, tripping, etc., to occur as they might cause injury, obstruction, irritation, decubitus, falling or other safety hazards.
- Do not use the product if you observe any signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Do not use product if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Products are supplied non-sterile and must be unpacked, cleaned and disinfected prior to first use.
- Adequate cleaning is a necessary requirement for subsequent effective disinfection.

- Follow the equipment and product-specific validated procedures for cleaning and disinfection.
- Use the equipment and accessories must be maintained and checked regularly according to the manufacturer's instructions.
- Additionally all applicable laws, standards and guidelines for reprocessing equipment should be followed.
- Use of personal protective equipment is the responsibility of the user.
- Deviation from the defined procedures within this instruction is the responsibility of the user.
- Manufacturer is not responsible for damages resulting from improper reprocessing. It rests with the user to validate their procedures and to comply with all applicable regulatory requirements.
- Use of accessories with the product is not recommended. Using accessories with the product other than the ones referenced might result in a measurement error or hazardous condition. Observe connector part color coding and/or markings on products to ensure adequate compatibility with monitor and other accessories.

- Do not use inappropriate and safe functioning of the product, check stability of connection with monitor and/or accessories initially and occasionally over time.
- Do not use inappropriate position or position of the product initially and occasionally over time to prevent entanglement, pressure, tripping, etc., to occur as they might cause injury, obstruction, irritation, decubitus, falling or other safety hazards.
- Do not use the product if you observe any signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Do not use product if you see any visual signs of damage. A damaged product should be discarded immediately. Never try to repair a damaged product, do modifications to product or use a product if repaired by others.
- Products are supplied non-sterile and must be unpacked, cleaned and disinfected prior to first use.
- Adequate cleaning is a necessary requirement for subsequent effective disinfection.

**Pre-Processing Instructions**
1. Ensure all pre-processing instructions are followed prior to cleaning and disinfection.
2. Follow cleaning and disinfection of device as soon as possible after use (recommended within 1 hour).
3. Transport devices via the institution's established transport procedure.
4. Remove excess gel or foam as possible after use by the manufacturer's device.
5. Disinfect device by cleaning and disinfected utilizing a minimum of one (1) cleaning method and one (1) disinfection method outlined below.

**Cleaning Instructions**
1. Ensure all pre-processing instructions are followed prior to cleaning.
2. Prepare the enzymatic / neutral pH detergent, utilizing tap water until a temperature range of 27 °C - 44 °C (81° F to 111° F) per manufacturer's instructions.
3. Immerse the device cloth saturated with the cleaning solution to manually clean all surfaces of the device including cracks and crevices for a minimum of 5 minutes. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lamden brush) to disinfined hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

4. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lamden brush) to disinfined hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.
5. Use a sterile, gauze saturated with purified water (PURW) at 27°C – 44 °C, manually rinse by wiping the device for 30 seconds. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh purified water each time.
6. Drying: Dry the device using a sterile, lint-free cloth.
7. If visible soil remains, repeat cleaning procedure until the device is thoroughly clean.

### Disinfection

**Intermediate level disinfection instructions – Option 1**
1. Prepare a 2% Glutaraldehyde disinfectant solution (CIDEX® or equivalent) according to the manufacturer's directions.
2. Using a sterile, soft cloth saturated with the disinfection solution, manually disinfect all contaminated surfaces of the device for a minimum of 15 minutes.
3. In conjunction with this step, use an appropriately sized brush (e.g. standard soft bristle toothbrush or lamden brush) to disinfined hard-to-reach areas paying particular attention to cracks and crevices ensuring a slightly wet surface is maintained at room temperature for the duration of this disinfection step.

4. Use a sterile, gauze saturated with purified water (PURW) at 27°C – 44 °C, manually rinse by wiping the device for 30 seconds. Repeat two (2) additional times for a total of three (3) rinse cycles, utilizing fresh purified water each time.
5. Drying: Dry the device using a sterile, lint-free cloth.

**Storage**
Storage temperature: -30 °C/-22 °F to +60 °C/+140 °F

**Disposal**
This device is not subject to disposal. Follow electrical and electronic devices compliance with WEEE or dispose per local and state environmental regulations. This product does not contain PVC.

### Technical description

- For technical description see the host device user guide or manual, or contact your local sales representative for more information.
- Not made with natural rubber latex.

### Ordering information

For further information, please contact your sales representative or visit [vyaire.com](http://vyaire.com).

Trademarks are the property of their respective owners.

©2019 Vyaire, Vyaire, the Vyaire logo and Vital Signs are trademarks or registered trademarks of Vyaire Medical, Inc., or one of its affiliates.

<span></span>	<span></span>	<span></span>	<span></span>
Vyaire Medical Oy	Kortteentiekuu 2	FI-02010 Helsinki, Finland	+1-833-327-3284
Customer Support	customersupport@vyaire.com		

100004806A - 2019-11

## Adapter for clips on conductive Electrodes for use in KISS™ System

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