

EKG Accessories User's Manual

1-Product introduction

- Product name: EKG Accessories (including Limb Clamp Electrode or Bulb Suction Electrode)
- Model: UU series
- Intended use: used for keeping electrodes fixed on the body.

2-Operation conditions

- Ambient temperature: 0 °C~ 40°C
- Relative humidity: 15%~85%
- Atmospheric pressure: 86kpa~106kpa

3-Storage/Packaging

- The products must be stored in its original packaging and within the storage conditions to maximize the storage life. The storage conditions are as follows:
 - 1、Ambient temperature: -10 to +55°C
 - 2、Relative humidity: 10 to 85%

4-Operation Procedures:

- Choosing applicable Limb Clamp Electrode or Bulb Suction Electrode to match the ECG Cable model of Electrocardiograph, according to the connector or interface between electrodes and ECG Cables.
- It is important to use the electrolyte containing product such as conductive gel to reduce the resistance value of the skin contacted, before using the ECG accessories.
- Place the Limb Clamp Electrode or Bulb Suction Electrode to the suitable position of the subject, and then connect the ECG Cable of Electrocardiograph to Limb Clamp Electrode or Bulb Suction Electrode, the electrodes providing three different kinds of connecting method, including Clamping, Snapping and Inserting. And then start the ECG monitoring.

5-Cleaning

- Clean or disinfect before applying to a new patient
- 70% isopropyl alcohol or alcohol solution can be used as a cleaning solution to clean the surface of the product.
 First apply the cleaning solution onto a clean and dry sponge pad. Use this sponge pad to wipe all the surfaces, and then dip another clean and dry sponge pad on sterile or distilled water. Use this sponge pad to wipe all the surfaces. Finally, use a clean, dry sponge pad to wipe all the surfaces of the wires.
- Note: the connector end pin shall not touch the cleaning solution; otherwise it will cause permanent damage to the electric conduction line of the conductor and the monitoring instrument.

6-Warning










- Disposal of the products shall comply with local regulation.

CAUTION:

Federal (U.S.) Law restricts this device to sale by or on the order of a physician.

Symbol explanation

Caution	Serial number	Manufacturer	Latex free	See the instructions	Waste Electrical and Electronic Equipment
---------	---------------	--------------	------------	----------------------	---

					
Date of manufacture	Catalogue number	Authorized representative in the European Community			
					



Unimed medical Supplies Inc

Bld#8, Nangang 3rd Industrial Park, Tangtou, Shiyao, 518108

ShenZhen, P.R. China

Obelis s.a.

Boulevard Général Wahis 53

1030 Brussels, BELGIUM

Tel: +(32) 2. 732.59.54

Fax: +(32) 2.732.60.03

E - Mail : mail@obelis.net



REF: Y07-M136

Rev.: A1

Date: 2018.08.02