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Instruction for use ECG-electrodes

Application / Warnings:

Derivation of bioelectric signals from the intact skin. The electrodes must be applied by trained medical personnel to the patient. Possibly occurring allergic reactions or skin sensitivities examine medically. Not suitable for use on newborn and premature infants! Use only with approved patient cables and / or adapters. Not be in contact with external electrical voltages. This product is a medical disposable and may be used only once. Electrodes in which the gel has dried may not be used. Consult a doctor immediately in case of accidental ingestion. In an electro-surgery, the ECG electrodes may not be placed as close to the surgical field as the neutral electrode (ground).

Directions for use:

1. Open the bag at the marked line
2. Setting the application points. Selected skin parties have to be cleaned of dense hair, cream, dirt or other interfaces. Do not apply to uncleaned skin, otherwise no reliable adhesion and signal leakage
3. Withdraw the electrode from the griptab by the protective film
4. Place the electrode and press it to the skin
5. Contact the patient lead, in case of pressure sensitive skin contact electrode prior to application to the skin

Duration of use: Check daily tightness and applied skin surface if the electrode is used for several days. The electrodes are not repositionable.

Storage: In their original packaging protects from the weather between -13 °C und +28 °C. Note shelf life. Consume partially used bags quickly, protect remaining electrodes by folding over the bag opening against moisture loss. Do not store the electrodes outside the original packaging (bag)!

Disposal: Dispose as household waste, PVC and latex-free, recycle packaging

We are not liable for damages caused by non-compliance with these instructions, abuse or caused by errors in the application of this product in any way.