Manual Aneroid Sphygmomanometer Blood Pressure Cuff Kit -

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This product collects and transmits human blood pressure signals by capturing the pressure on the cuff bladder caused by blood vessel contraction and dilation. It is designed for non-invasive blood pressure measurement in adult users with an arm circumference of 22-32cm, suitable for home use, hospitals, clinics, and other medical settings.

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1.3 Product Features / Produktmerkmale

- Adult-sized red cuff with a detachable TPU bladder, ensuring easy cleaning and replacement.
- Equipped with a handheld pressure display gauge with a high-contrast, easy-to-read dial (measurement range: 0-300mmHg) for precise readings.
- Features a single-tube design with a 45cm PVC connecting tube, ensuring stable air transmission.
- The inflation bulb is made of crack-resistant, non-sticking, high-density, hypoallergenic, and latex-free PVC, which is sturdy, durable, and safe to use.
- Comes with a large screw-type valve for precise deflation control, enabling smooth pressure adjustment.
- Lightweight and portable, accompanied by a pocket bag and storage bag for convenient carrying and organized storage.
- No electronic equipment or battery required; relies on manual inflation and deflation to obtain professional and accurate blood pressure data.

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- Check the product integrity: Ensure all components (cuff, TPU bladder, pressure gauge, inflation bulb, connecting tube, valves, bags) are intact, free of damage, aging, or air leakage. Confirm the detachable bladder is securely connected.
- Verify the user's arm circumference: Ensure it falls within the 22-32cm range; do not use if the arm circumference is outside this scope.
- Prepare the user: Ask the user to sit quietly for 5 minutes before measurement, avoiding exercise, smoking, drinking alcohol, or consuming caffeinated beverages. Remove thick clothing from the measurement arm to keep the skin dry and clean.
- Choose a suitable environment: Use the product in an area with a temperature of $0^{\circ}\text{C} \sim 45^{\circ}\text{C}$ and relative humidity $\leq 95\%$, avoiding direct strong light, severe vibration, or extreme temperature changes.

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B. Cuff Application / Manschettenanlegung

- Align the "ARTERY" marker on the inner side of the cuff with the user's brachial artery (above the inner elbow).
- Wrap the cuff around the upper part of the user's upper arm, ensuring the bladder is positioned correctly against the skin. The tightness should allow two fingers to be inserted between the cuff and the arm—avoid excessive tightness (which may impair blood circulation) or looseness (which may cause inaccurate measurements).
- Connect the PVC tube of the cuff to the handheld pressure gauge and inflation bulb, ensuring tight connections without air leakage.

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C. Measurement Operation / Messbetrieb

- **Manual inflation:** Squeeze the PVC inflation bulb repeatedly to inflate the cuff until the pressure gauge pointer reaches a value 30-40mmHg higher than the estimated systolic blood pressure (as recommended by medical professionals).
- **Precise deflation:** Slowly rotate the screw-type deflation valve to control the deflation rate, allowing the pressure gauge pointer to drop steadily at a rate of 2-4mmHg per second.
- **Reading data:** Observe the high-contrast dial of the pressure gauge. Record the systolic blood pressure (the value when the pointer first jumps significantly, corresponding to the first clear Korotkoff sound) and the diastolic blood pressure (the value when the pointer stops jumping or the Korotkoff sound disappears).
- **End measurement:** After recording the data, fully open the deflation valve to release all air from the cuff. Remove the cuff from the user's arm, disconnect the components if necessary, and store all parts in the storage bag.

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- This product is exclusively for non-invasive blood pressure measurement in adults. Do not use it for neonatal, infant, or pediatric users, or for any purpose other than its intended use.
- Do not connect the cuff or inflation system to intravascular infusion systems, as this may pump air into blood vessels and cause severe injury to the user.
- During measurement, minimize movement of the user's arm and the cuff. Avoid touching, pressing, or moving the cuff to ensure accurate measurement results.
- For frequent or long-term monitoring (more than 4 hours continuously), regularly check the measurement site for signs of impaired blood circulation (such as discoloration, numbness, or coldness). Change the cuff's wrapping position every 2 hours to prevent tissue damage.
- Do not wrap the cuff around an arm with intravenous infusion, damaged skin, ulcers, burns, or impaired tissue. Ensure the fleece or hook surface of the cuff closure does not come into direct contact with the skin to avoid irritation or injury.
- If the cuff width exceeds the length from the user's armpit to the elbow, do not use it on the upper part of the upper arm; choose an appropriate cuff size instead.
- Before each use, inspect the product components: Replace the cuff immediately if signs of aging, tearing, cracking, or loose wrapping are found. Check the air tightness of the tube and bulb—if air leakage occurs, stop using and repair or replace the faulty part.
- Do not inflate the cuff when it is not wrapped around the arm, as excessive pressure may damage the bladder or gauge.
- This product requires regular calibration (recommended annually) to maintain measurement accuracy. Calibrate it at an authorized service center or by a professional technician.
- Keep the product out of reach of children to avoid accidental ingestion or injury.

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A. Cleaning Supplies / Reinigungsmaterialien

- Enzyme cleaner (e.g., ENZOL from the US or Cidezyme from the UK)
- 10% bleach solution
- Separate spray bottles
- Dust-free cloth
- Soft-bristled brush
- Distilled water

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- Disassemble the product: Detach the TPU bladder from the cuff and disconnect the connecting tube from the gauge and bulb.
- Pre-treat stains: Spray the enzyme cleaner evenly on the cuff, bladder, tube, and bulb. For dried stains, let the cleaner sit on the surface for 1 minute to soften the dirt.
- Wipe and scrub: Wipe smooth surfaces (such as the gauge shell and bulb) with a dust-free cloth. Use a soft-bristled brush to gently scrub the cuff fabric, bladder, and tube to remove visible stains.
- Rinse thoroughly: Rinse all cleaned components with a large amount of distilled water to remove residual cleaner and dirt.
- Dry naturally: Place the components in a well-ventilated area to air dry completely. Do not use a hair dryer or high-temperature equipment to dry, as this may damage the material.

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- a. After cleaning, spray the 10% bleach solution evenly on the cuff, bladder, tube, and bulb until saturated.
- b. Let the solution sit for 5 minutes to ensure effective disinfection.
- c. Wipe off excess bleach solution with a dust-free cloth.
- d. Rinse thoroughly with distilled water to remove any residual bleach.
- e. Air dry completely before reassembly and storage.

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D. Notes / Hinweise

- a. Do not soak the handheld pressure gauge in any liquid; only wipe its surface with a damp cloth to prevent water damage to internal components.
- b. Ensure all components are completely dry before reassembly to avoid mold growth.
- c. Replace the TPU bladder regularly (recommended every 6-12 months) for optimal hygiene and performance.

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Lagerungsanforderungen

A. Transport Requirements / Transportanforderungen

- The transport vehicle must be clean, hygienic, and well-sealed to protect the product from rain, snow, dust, and dirt.

- During transport, avoid severe collisions, pressure, or vibration to prevent damage to the pressure gauge, bulb, or other fragile components.
- Do not transport the product with toxic, harmful, corrosive, or flammable items.

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B. Storage Requirements / Lagerungsanforderungen

- Storage Environment: Store in a dry, well-ventilated warehouse with a temperature of $-5^{\circ}\text{C} \sim 40^{\circ}\text{C}$ and relative humidity $<85\%$.
- Storage Conditions: Keep the product away from direct sunlight, heat sources, and humid areas. Store it in the provided storage bag to prevent dust accumulation and component damage.
- Shelf Life: 2 years from the production date (see the product label for the specific production date).
- During storage, check the product periodically (every 3 months) for signs of aging, mold, or air leakage.

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8. Symbol explanation:

Symbols	definition	Symbols	Definition
	Note investigation, please consult the documentation that came with		Product complies with the EU Medical Devices Directive 93/42 / EEC of identity
	Waste Electrical and Electronic Equipment	Rx ONLY	Federal (U.S.) Law restricts this device to sale by or on the order of a physician
	Storage temperature limits		serial number
	Storage humidity limits	REF	Catalogue number
	Large oxygen pressure limits		Company logo and Do not discard
	Latex-free		EU authorized agent
	Non-sterile packaging		manufacturer

	Please refer to the instruction manual		Manufacture date
		IPX2	Waterproof level

9. Manufacturer and EC Representative:

	Name: Hunan Greatmade Medical Tech Limited Add : NO.101, 1st floor, A2 building, 1st Phase, Changsha E center, No.18 Xiangtai road, Liuyang Economic Development zone, Changsha, Hunan Province, China SRN: CN-MF-000042754 Tel :86-731-83167851 Website: www.gmmedical.net Email: sales@greatmade.com.cn
	Name : SUNGO Europe B.V. Add: Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands SRN: NL-AR-000000247

10. Revision information

Rev.	Revision description	Revised date
A.0	First edition	2025/8/12
A.1	Includes English, German language	2026/1/27